

Bluetooth & Voice Notification



WHEELCHAIR DOCKING SYSTEMSTM




Apple App Store




Google Play Store

How to Download the EZ Lock App


App Images for Presentation below are from the Google Play Store

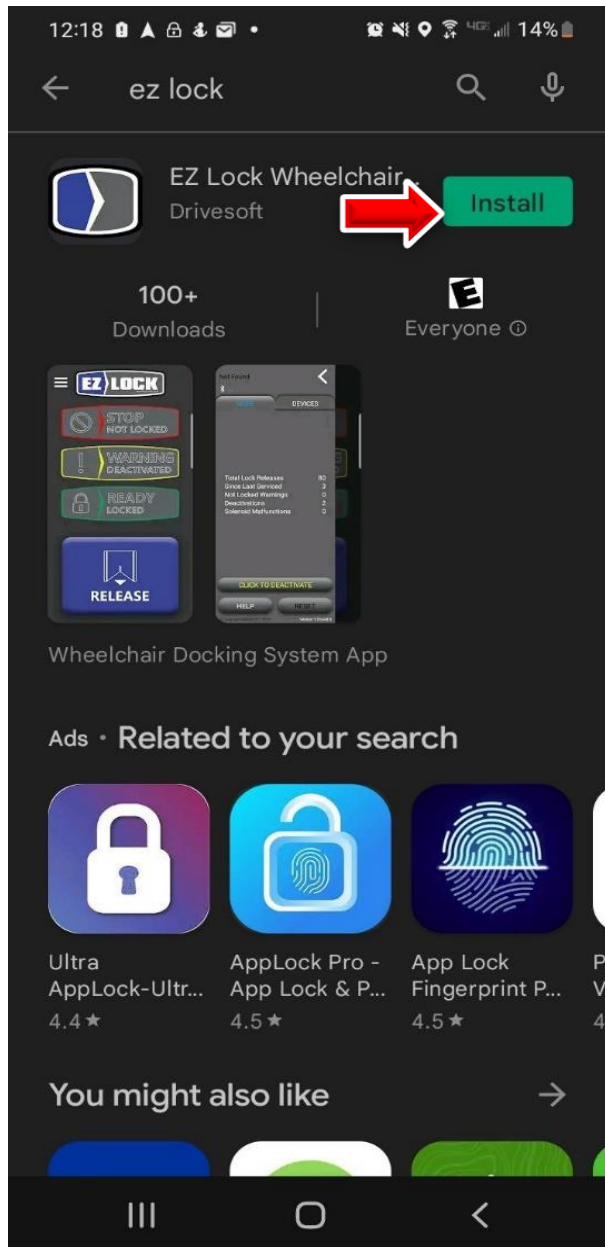


Open Google Play or Apple App Store Search EZ Lock - Press Install

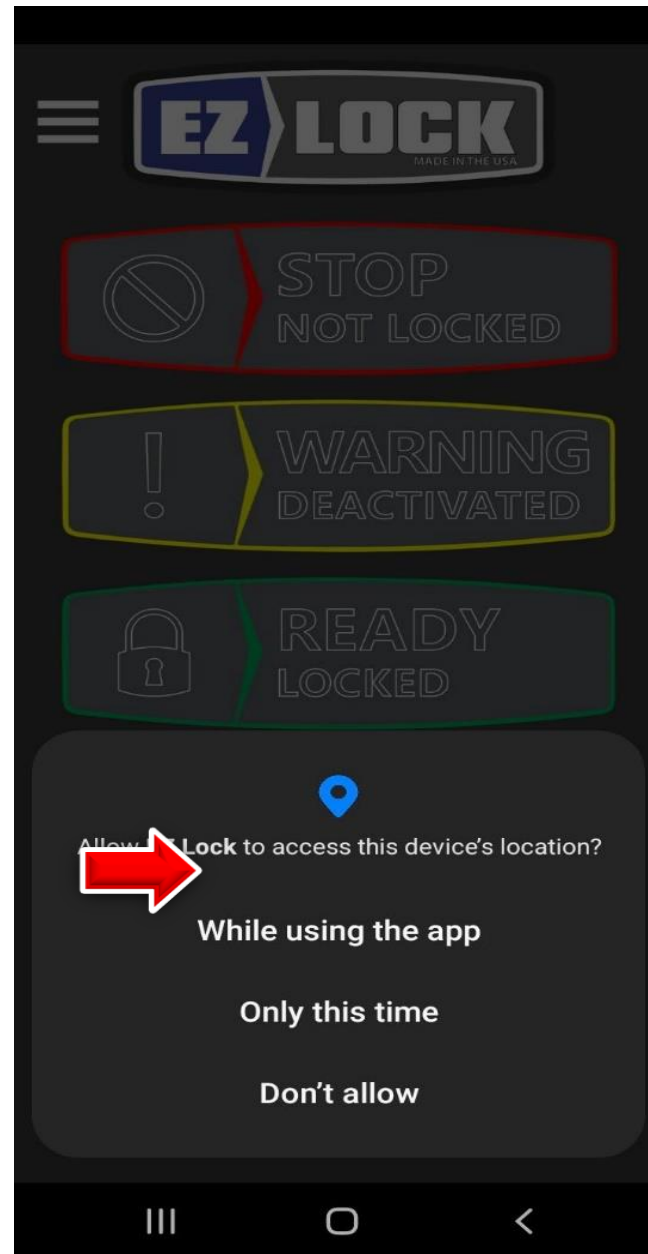


While using the app

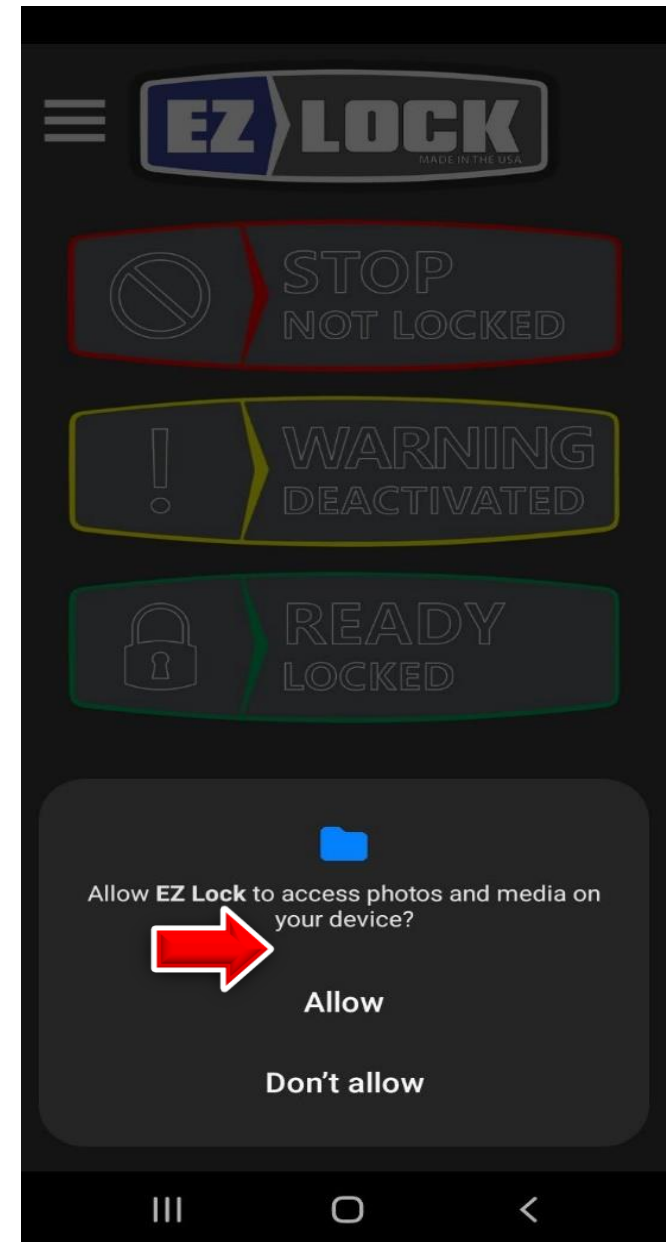


Allow App access to phone

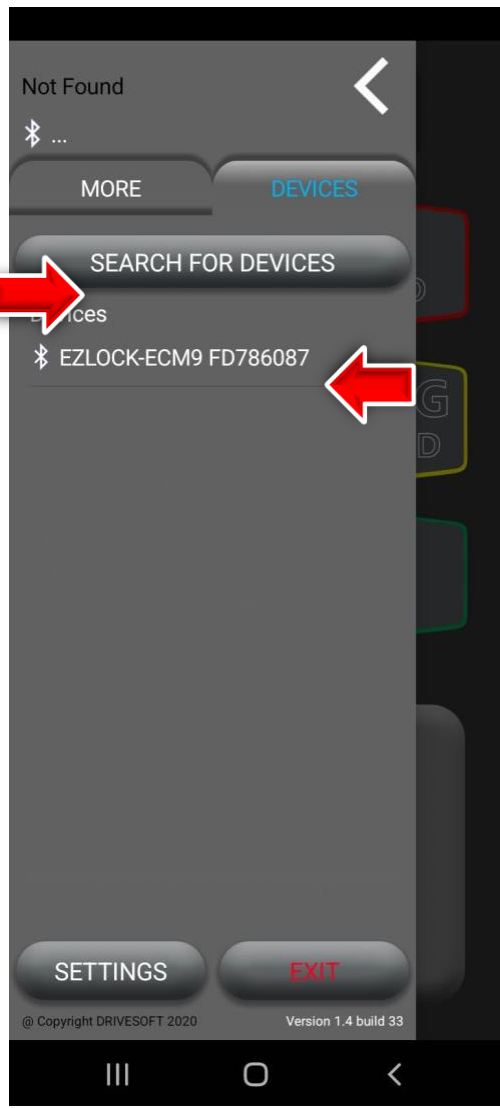

Search for
Devices

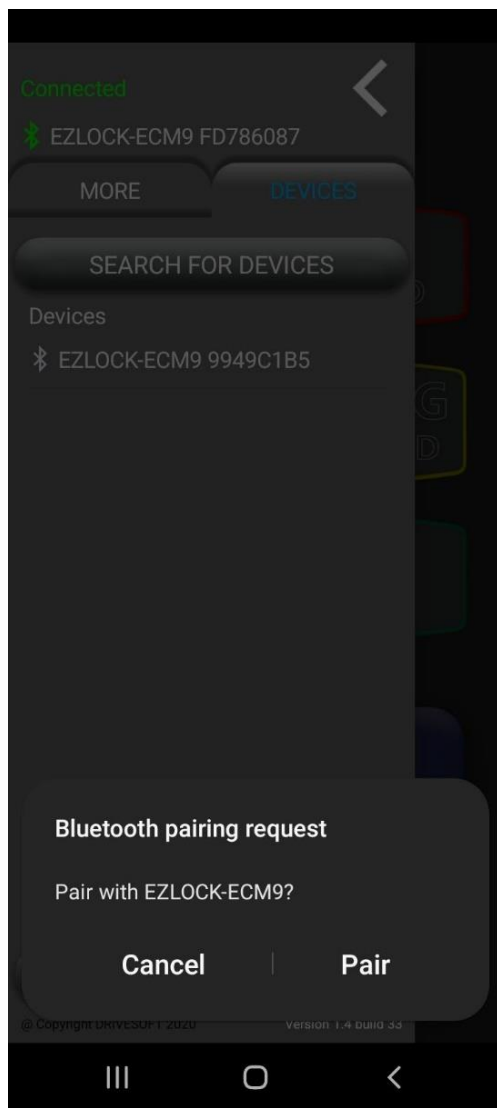


Pair

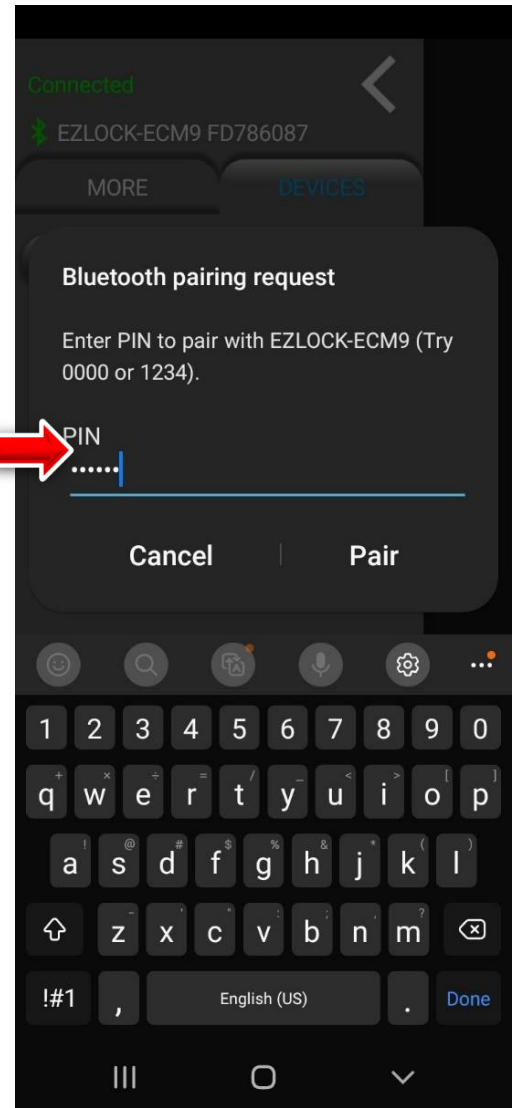

Pairing PIN
123456

Pairing the phone to ECM9 & Dealer Registration

NOTE: Register the Dealer information using the clients phone. Not the shop tablet.

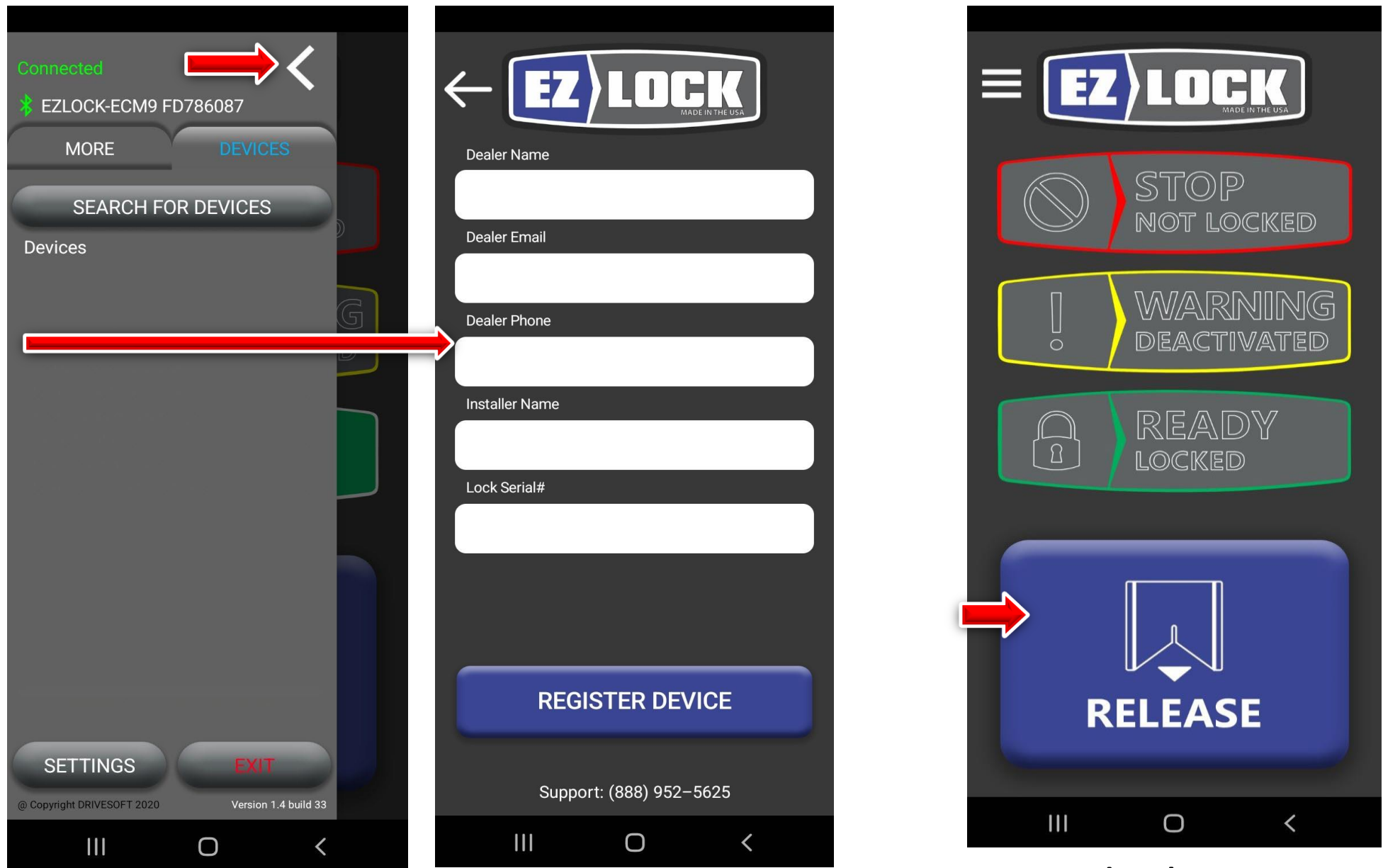


Dealer Information


Paired when RELEASE
Button is BLUE

Setup & Client Registration

NOTE: Register the clients information using the clients phone. Not the shop tablet.

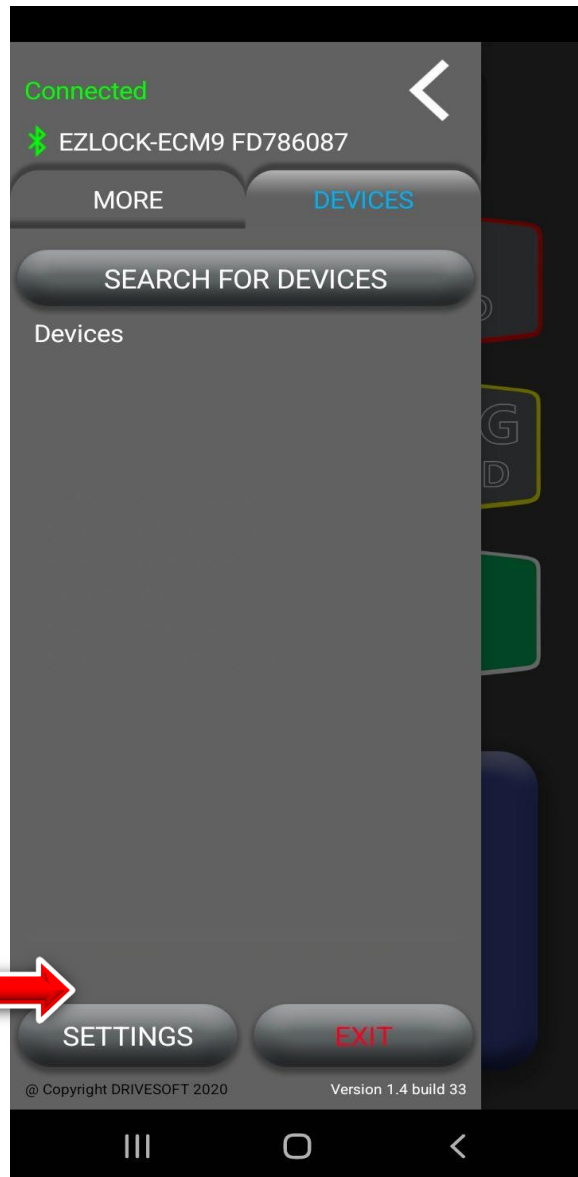

Settings

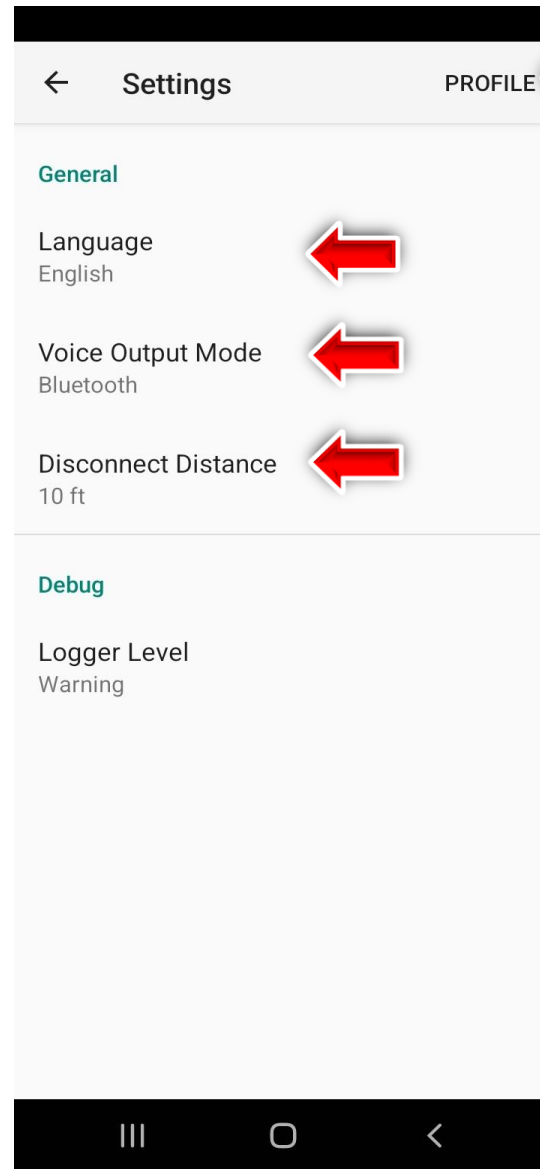


Profile

The screenshot shows the 'Client Information' registration form. At the top, there's a back arrow and the 'EZ LOCK' logo with 'MADE IN THE USA' underneath. The form contains several input fields, each with a red arrow pointing to it: 'Full Name', 'Street Address', 'City, State, ZIP', 'Phone Number', 'Email', and 'Password'. Below these fields is a large blue button labeled 'REGISTER'. At the bottom, there's a link 'Already have an account? Login here' and a support number 'Support: (888) 952-5625'.

Client Information

4 Levels of Lock Status Indicators – In App, Dash LEDs, Voice & Buzzer

All status conditions below are recorded & stored in the ECM9

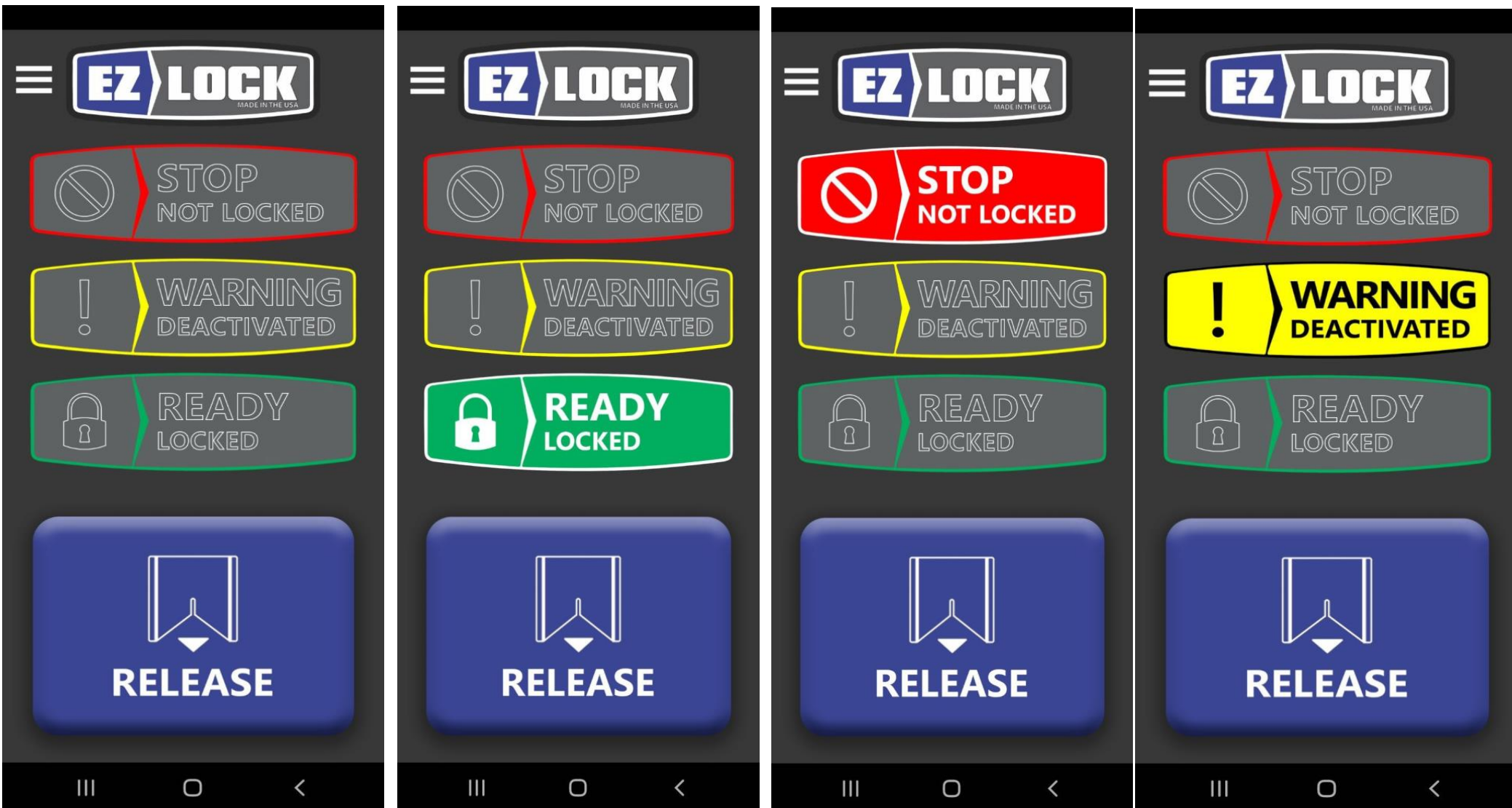

Ignition OFF
No LED
wheelchair
Can be **RELEASED**


Ignition ON
GREEN Led
wheelchair
LOCKED

Ignition ON
RED Led
wheelchair
NOT LOCKED

Ignition ON
YELLOW Led
EZ Lock is
DEACTIVATED

Counter Data

Counter data is stored in the ECM.

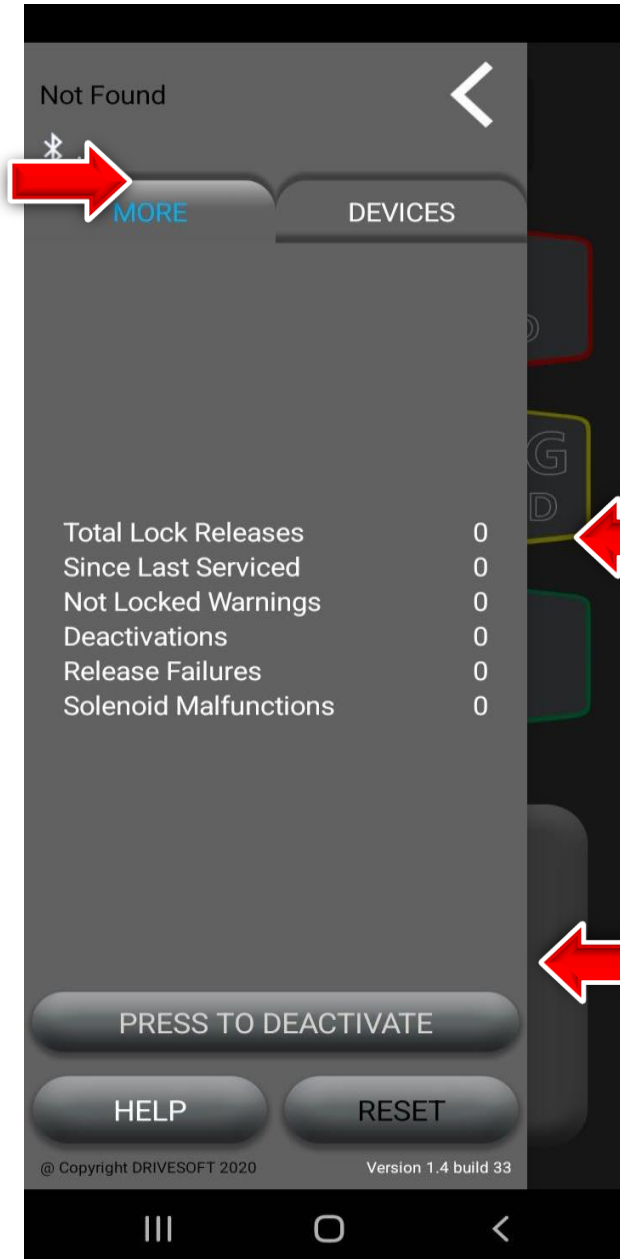

* You can pair a new phone and still access your data under Setting – More tab

* ECM records every function status of the docking system in real time:

- Total Number of Lock Releases since install & since last service

- Every Stop Warning Every Deactivation

- Every time the wheelchair did not release

- If there is a solenoid problem